


# Syncing VS4 Testing Reports to your CNS VS Account

1

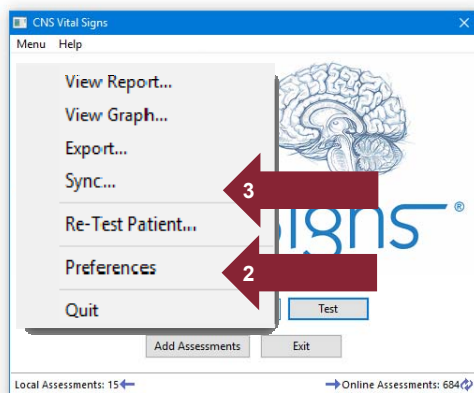

## To Set-up Sync


- 2) Open the CNS Vital Signs Application... CLICK MENU ...CLICK PREFERENCES to configure the SYNC function

## To Launch a Data Sync

- 3) Open the CNS Vital Signs Application... CLICK MENU ...CLICK SYNC to upload and/or download patient reports

Multiple testing computers in multiple locations e.g., telemedicine, ACO, etc. can also have their results backed-up to CNSVS Online. Testing results can be remotely accessed by clinicians with user permissions.

2

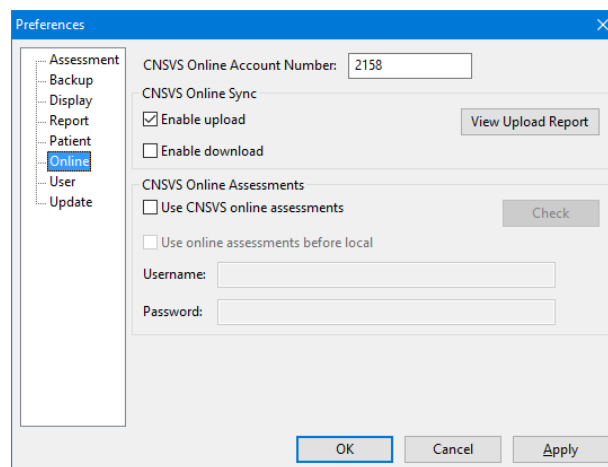


## Configure SYNC Functionality

CLICK the ONLINE tab

- Enter your account number.
- CHECK the 'Enable upload' box to back-up testing data.
- CHECK the 'Enable download' to replicate the testing data or download remote testing results e.g., telemedicine.

**Click Apply; Click Ok; Exit CNS Vital Signs**

3

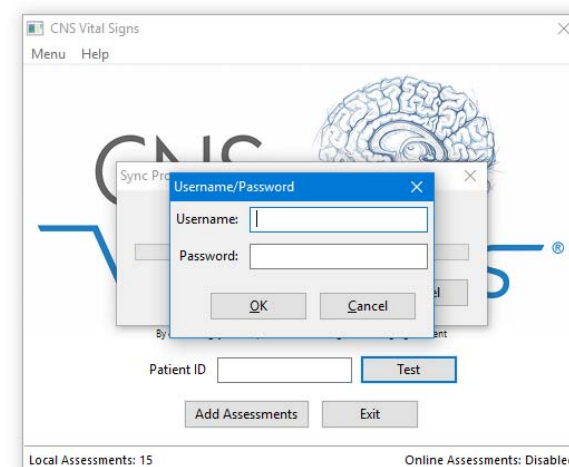

## Activate a Data Sync

Insert Username and Password to SYNC your LOCAL Software Testing Data to your CNS Vital Signs Account

*NOTE: To SYNC Your Data to a CNS Vital Signs Web Account you must FIRST have the CNSVS Support Team activate your CNSVS Online capability.*

# How do I SYNC my testing data?

